



## **TOPICAL USE CREAM WITH HYALURONIC ACID**

### **INDICATIONS**

Eudena Med is a topical cream based on hyaluronic acid, effective to treat superficial skin wounds, such as traumatic wounds and abrasions, and to treat minor burns, minor cuts or excoriations. Thanks to the hyaluronic acid, Eudena Med creates a barrier-effect film that protects skin from dehydration caused by skin lesions. Moreover, its high moisturizing power creates favourable conditions for tissue regeneration.

### **COMPOSITION**

Water, Glycerin, Steareth-2, Glyceryl stearate, PEG-8 distearate, Butyrospermum parkii butter, Panthenol, Cetearyl ethylhexanoate, Ethylhexyl palmitate, Diisopropyl adipate, Simmondsia chinensis oil, Tocopheryl acetate, Cetyl alcohol, Steareth-21, Propylene glycol, Dimethicone, Caprylyl glycol, Ethyl linoleate, Ethyl linolenate, 1,2-hexanediol, Sodium hyaluronate, PEG-7 glyceryl cocoate, Hydrolyzed glycosaminoglycans, Lecithin, Panicum miliaceum extract, Tropolone, Tocopherol, Benzyl alcohol, Ascorbyl palmitate, Sodium hydroxide, Sodium benzoate, Potassium sorbate, Citric acid.

### **POSODOGY AND HOW TO USE IT**

Wash hands well before use. Clean the skin using a suitable sanitizer before applying the product. Evenly spread a small amount of product over the affected area several times a day. The cream can be used alone or with an appropriate dressing, such as gauze, bandages, patches, and non-adherent wound dressings. In this case, change the dressing according to the doctor's and/or pharmacist's instructions.










### **WARNINGS AND PRECAUTIONS FOR USE**

- For topical use only.
- Keep out of the reach of children.
- In case of concomitant drug treatments, consult the doctor.
- No studies have been conducted in pregnancy and lactation. Therefore, in these cases, use the product only after consulting a doctor.
- Product functionality is guaranteed if the primary packaging is intact. Do not to use the device if the packaging is damaged or not fully intact when first opened.
- Do not use it after the expiry date specified on the package.
- Do not let it come into contact with the eyes. In case of accidental contact, thorough washing with warm water is recommended.
- Do not use in case of deep wounds, second and third degree burns
- Do not dispose the tube as household waste after use.
- Do not expose the areas of the body treated with Eudena Med to sunlight (risk of photosensitivity).

## STORAGE

Validity after first opening: 3 months. Store in a cool and dry place, away from heat sources, at a controlled room temperature not exceeding 25°C. Carefully close the tube after each use. The expiry date applies if the product is properly stored and its package is not damaged. Do not dispose of the product or its package as household waste after use.

Eudena Med is sold in a 25-g tube.

Symbols used on the package					
	Manufacturer		Do not use if package is open or damaged		Medical device in compliance with Directive 93/42/EEC as amended
	Batch number		Please consult the instructions for use before use		Caution
	Expiry date		Non-sterile medical device		Store at temperatures not above 25°C

 **AESCULAPIUS** farmaceutici s.r.l. - Via Cefalonia, 70 - Brescia

REF 913290207184

REV 01/2021

**AESCULAPIUS**  
FARMACEUTICI