

Eutuss

Isotonic solution for inhalation with essential oils

Eutuss is a medical device indicated for aerosoltherapy.

Qualitative composition

0.9% sodium chloride, *Pinus mugo*, *folium Aetheroleum*, *Lavandula angustifolia*, *summitas cum floribus Aetheroleum*, *Helichrysum italicum*, *flos Aetheroleum*, *Lavandula x intermedia*, *flos Aetheroleum*, vegetable glycerin, propylene glycol, polysorbate 20, water.

Essential oils are extracted in a steam stream and no other extraction methods are used, moreover they are not modified by adding or removing molecules.

Product type

Medical device.

Packaging

Case containing 5 vials of Eutuss, isotonic solution with essential oils.

One vial contains 5ml of solution.

Posology

The single dose of Eutuss is 2,5 ml.

The content of a 5 ml vial allows 2 daily treatments.

Your doctor will indicate the most appropriate individual dosage.

Indications

Eutuss is indicated in patients in the adult and pediatric age to help the elimination of excess mucus. It attenuates irritation and inflammation-related symptoms in respiratory tract, even when accompanied by coughing, and cold symptoms such as nasal congestion and nasal mucosa irritation even in case of allergic rhinitis or sinusitis. It promotes fluidization of the secretion by facilitating breathing.



Warnings and precautions for use

- The solution is intended for inhalation use only. It is not intended for oral use. Not injectable.
- Check the expiry date on the package before use. Do not use after the expiry date.
- Do not use if the package is open or damaged.
- The product, which is not immediately used after the first opening, must be kept in a refrigerator and used within and not more than 12 hours.
- Do not freeze.
- Keep out of sight and reach of children under three years of age. The product contains small parts that can cause suffocation.
- Treatments with pediatric patients should be performed under the supervision of an adult.
- Do not use in case of known hypersensitivity to one or more components contained in the solution.
- Patients with tendency to breathlessness or hypersensitive should use Eutuss only under prescription.
- Do not use, unless under medical advice, below 2 years of age, as in this age range the physiological characteristics of the respiratory tract lead to a limited capacity for drainage of the bronchial mucus.
- Do not dilute the solution and do not use the product in combination with other products because it may alter their action.

How to use

Inhalation of the solution can be done with a device for aerosol, using a mask, actuator or nasal cannula, following the directions for use provided by the device for aerosol manufacturer.

- Remove a vial of Eutuss from the strip of 5 vials present in the pack.
- Shake the vial to facilitate the homogeneity of the solution.
- Open the vial by turning the cap, do not throw the cap as it is necessary to close the vial.
- Use half the contents of the vial for a single treatment, pouring the solution into the nebulizer until the notch on the vial.
- Turn on the device for aerosol by dispersing the solution in the environment for about 10 seconds before inhaling.
- Inhale the nebulized solution according to the operating instructions supplied by the device for aerosol manufacturer, for a period of 10 minutes.

- Close the vial with the cap. The remaining solution must be stored in the fridge and used for any subsequent daily treatment within and not more than 12 hours after the first opening.



At the end of treatment, a white residue without any active function can be seen in the ampoule of the device for aerosol. It must be removed according to the cleaning operations provided in the operating instructions of the device for aerosol manufacturer.

Side effects

Allergic reactions are possible in people with intolerance to one or more components contained in the solution. During the inhalation of the isotonic solution, in very sensitive patients suffering from bronchial asthma or bronchial hyperreactivity, a reversible bronchodilation may occur. In these cases, discontinue therapy and contact your doctor or pharmacist.

It is important to report to your doctor, pharmacist or your pharmacist if you notice any side effects not listed in this leaflet.

Shelf-life

See the expiry date on the package.

How to store

Store at room temperature, away from direct sunlight and sources of heat, in the properly closed package.

The expiry date refers to the product properly stored in unopened package.

Disposal methods

Do not dispose of the container after use.

The product must be disposed of in accordance with local laws on municipal waste.

AESCULAPIUS FARMACEUTICI S.r.l.

Via Cefalonia, 70 – 25124 Brescia

Symbols used on the packaging			
	Manufacturer		Batch number
	Expiry date		Do not use if the package is open or damaged
	Read the instructions before use		Not sterile Medical Device
	Medical device complying with 93/42/CEE Directive and further modifications		Warning

Rev. 01/2019

FI9129682650500.02